RESPIRATOR SEAL CHECKS GREAT PLAINS CENTER FOR A SPICULTURAL Health

Respirators can provide protection from respiratory hazards. To provide effective protection, a respirator must fit the individual.

To ensure a respirator fits:

- ✓ Participate in an annual respirator fit test
- ✓ Ensure there is nothing blocking the seal (e.g., facial hair)
- ✓ Perform a seal check EVERY TIME a respirator is put on

After each use, clean respirator and store in an airtight container.



Perform a positive & negative pressure check for BOTH types of respirators.

POSITIVE PRESSURE CHECKS

Exhale while covering respirator

Seal Achieved: If respirator slightly expands

NEGATIVE PRESSURE CHECKS

Inhale while covering respirator **Seal Achieved:** If respirator slightly collapses

FOR FILTERING FACEPIECE:

If Air Leaks In/Out...

Readjust nosepiece with both fingers. Repeat check until seal is achieved.

FOR ELASTOMERIC:

If Air Leaks In/Out...

Readjust the straps.

Repeat check until seal is achieved.

Follow the manufacturer's instructions for seal checks on specific respirators.